



# What has changed?

DreamStation	DreamStation 2
Navigation dial with color screen and tile views	Color screen with 2 capacitance touch zones for simple interactions
Navigating in and out of sections to set up device	Guided 3 step prescription set up (Mode, Pressure, Tube Size) Other defaults assumed and adjustable in Care Orchestrator (FLEX, Ramp Plus time, Heated Tube temperature, Auto ON/OFF, Opti-Start)
Fixed Ramp (pressure ramps up) or Smart Ramp (auto algorithm kicks in if event takes place) Pressures: 4 cmH2O to MIN pressure in 0.5 cmH2O increments Time: 5-45 min in 5 min increments Requires push of Ramp button every night to initiate	Ramp Plus - constant pressure delivered unless an event takes place causing auto algorithm to take over Pressures: LOW (4cmH2O), MEDIUM (6cmH2O), HIGH (8cmH2O), MAX (10cmH2O) Time: 15, 30 (default), 45 min adjustable in Care Orchestrator Once Ramp Plus is set there is no need to press a button again to initiate. It automatically starts at selected pressure with each therapy session.
Ramp pressure adjustment when therapy is off	Live adjustment of Ramp Plus pressure (along with humidification) while therapy is running
Compliance data in different locations	All patient data in Care Orchestrator or through scrolling view when therapy turned off
Modem calls every 24 hours	Modem calls every 24 hours + on demand modem calls for Rx push and data pulls (device plug in or push from Care Orchestrator)
Performance check via patient menu	Performance check with every device plug in